



Upgrading to ShowCase 6.7

Overview

SHOWCASE® 6.7 has several new features:

- Mark studies as Read (or Unread)
- Change Patient Name
- A Patient Date of Birth column has been added to the Study List
- Customizable Study List columns
- Customizable Study List font size

Doing The Upgrade

The upgrade is straightforward, but it does require that you have Administrator privilege to do it.

1. Download the new ShowCase DICOM Viewer installer from here: <https://www.triltech.com/download.html>
2. Uninstall your existing ShowCase 6.x software. All of your data, configuration, and license settings will still be on the computer.
3. Run the installer you downloaded in step 1.
4. Start ShowCase

SHOWCASE® is a registered trademark of Trillium Technology, Inc.